SDC-G5-Shank Instructions for use – ENG

Items references:

- 1990 (G5-Shank pink)
- FS-150 (G5-Shank Ohead blue)

To be used in combination with the following items references:

- 1800/UF (SDC-CombiStrip Ultra Fine 6µm, two sides diamond coated)
- 1800/F (SDC-CombiStrip Fine 15µm, one side diamond coated)
- 1500 (G5-ProCut)
- 1310/1 (G5-ProLign 0.1 mm, one side diamond coated)
- 1310 (G5-ProLign 0.1 mm, two sides diamond coated)
- 1320/1 (G5-ProLign 0.2 mm, one side diamond coated)
- 1320 (G5-ProLign 0.2 mm, two sides diamond coated)
- 1330 (G5-ProLign 0.3 mm, two sides diamond coated)
- 1340 (G5-ProLign 0.4 mm, two sides diamond coated)
- 1350 (G5-ProLign 0.5 mm, two sides diamond coated)
- 1600/UF (G5-ProShape Ultra Fine 6µm, one side diamond coated)
- 1600/F (G5-ProShape Fine 25µm, one side diamond coated)
- 1600/MF (G5-ProShape Medium Fine 40µm, one side diamond coated)
- 1115/1 (G5-ProStrip 15µm, one side diamond coated)
- 1115 (G5-ProStrip 15µm, two sides diamond coated)
- 1125/1 (G5-ProStrip 25µm, one side diamond coated)
- 1125 (G5-ProStrip 25µm, two sides diamond coated)
- 1140/1 (G5-ProStrip 40µm, one side diamond coated)
- 1140 (G5-ProStrip 40µm, two sides diamond coated)
- 1106 (G5-UltraSoft 6µm, two sides diamond coated)

Compatible with all EVA oscillating system handpieces on the market with an instrument insertion diameter $\varnothing=2.30\pm 2.35$ mm.

Particularly indicated to be used with the following handpieces, items references:

- 1971 (NSK Ti-Max X55L, optic)
- 1972 (NSK Ti-Max X55, non-optic)
- 3001 (Nose Cone) + 1920 (SDC O-Head)

1. Purpose

Mechanical connection between the oscillating instrument G5 attack and the reciprocating handpiece head.

2. Introduction

In innovative dentistry, it is common to use oscillating mechanical instruments. If these are equipped with a G5 attack, it is necessary to use a connection element between them and the reciprocating handpiece.

3. General information

The shanks family, consisting of ergonomic plastic shanks are used for the connection between the oscillating instrument G5 attack and the reciprocating handpiece head:

- Material: high resistant polymer
- Sterilisable and re-usable

The guidelines implemented and regularly applied for the design and production of these file devices, are imposed by the harmonized international standards.

4. How to install the instrument

- Insert the shank from its end into the front part of the handpiece head taking care to choose the right shank indifferently of the supplied handpiece:
 - All EVA oscillating system handpieces: pink shank (item ref. 1990)
 - SDC O-Head: blue shank (item ref. FS-150)
- Slide the shank until it is fully inserted and verify it is locked into the handpiece
- At the bottom side of the shank there is a little stopper which prevents the instrument from slipping out: insert the instrument

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from the bottom side into the shank and push upwards until it clicks fit

5. Operating Principles

- Once the tool is inserted into the handpiece head be sure it is fully and firmly fixed
- Switch on the handpiece by turning on the micro-motor
- Set and bring the instrument to the recommended speed before applying it on the tooth
- With the instrument in motion enter the inter-proximal space to be treated with a cervical motion
- Proceed with the treatment paying attention to the speed and the pressure applied
- Once the treatment is done remove the instrument still in movement from interdental space
- Switch off the oscillating handpiece by turning off the micro-motor

6. Principles of safety

- Provide sufficient irrigation: at least 50 mL / min
- Do not exceed the recommended speed:
 - Handpiece NSK Ti-Max X55 / X55L: 40'000 rpm = 10'000 stk. / min,
 - Red Nose Cone + SDC O-Head: 20'000 rpm = 5'000 stk. / min
 - All other EVA oscillating system handpieces, refer to manufacturer instructions
- Do not exceed the recommended pressure: 1N ÷ 3N (max. 3N)
- Do not stop the motion of the instrument while the same is in the interdental space
- In case of overcrowding open the interdental space using a wedge in the cervical area
- Do not bend, warp or leverage the instrument
- Not observing these precautions leads to a high risk in terms of safety and can lead to the possible breakage or damage of the instrument
- SDC accepts no liability for improper use of its products

7. Warnings and Precautions

- The instrument is for exclusive professional use,  mainly for dentists specialized in Dentistry
- Do not bend, warp or leverage the instrument over the limit: the use of excessive force (leverage with the handpiece) can lead to premature breakage
- Excessive lateral swerve or obstruction during the oscillation movement increases the risk of instrument damage
- Do not exceed the 20'000 oscillations per minute to avoid breakage of the instrument
- Temperatures over 180°C are to be avoided in order to not reduce the instrument lifetime
- Avoid forces exceeding 3N because in this case you risk damaging the instrument
- The consumption of the instrument lifetime until its worn out, replacement and disposal is an exclusive decision of the physician according to his operative needs
- Verify instrument integrity before reutilization: absence of small cracks, breakages and/or deformations on plastic part, diamond presence on metal band
- Be careful not to injure the soft tissues, especially when you cross the equator of the tooth
- To ensure traceability during the instrument lifetime we recommend to preserve the packaging
- On the product packages SDC lists the references that allow you to trace the entire production cycle of each product. In case of detected problems of a constructive nature of SDC product, we ask you to return it, in its full packaging, to be able to trace the cause of the problem that we will report

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8. Maintenance

- The instrument is supplied non-sterile: sterilize before the first use
- In order to preserve the state of the instrument, after each use it must be cleaned of any residues, disinfected and sterilized

9. Cleaning

- Disinfection can take place in an ultrasonic cleaning device containing a liquid degreaser and / or a chemically inhibited disinfectant
- The cleaning of the instrument after each use can be done by brushing the diamond part provided that the brush bristles are not too rigid to avoid damaging metal strip, diamond and polymer

10. Sterilization

- Use single or fractionated vacuum autoclave and vacuum drying
- The hot air sterilizers are not suitable for diamond instruments
- Sterilization must follow validated procedures in compliance with European standard EN 13060:
 - B Standard 134 (T=135.5°C, P=2.16bar, t_{Plateau}=4', t_{Drying}=15', t_{Total}=30'÷40')
 - B Prion 134 (T=135.5°C, P=2.16bar, t_{Plateau}=18', t_{Drying}=15', t_{Total}=44'÷54')
 - B Standard 121 (T=122.5°C, P=1.14bar, t_{Plateau}=15', t_{Drying}=20', t_{Total}=50'÷60')

11. Contraindications

- Use only as described above to avoid incurring drawbacks due to the carelessness and distraction
- Do not use on patients suffering from indisposition and / or diseases of any nature and / or type
- Do not disperse in the environment
- The instrument gets in contact with biological tissues during its usage: dispose of the instrument in accordance with the destination country legislation
- Keep out of the sight and reach of children
- Do not swallow

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